
MacBrud Corporation – Instructions for use SpecBoard CE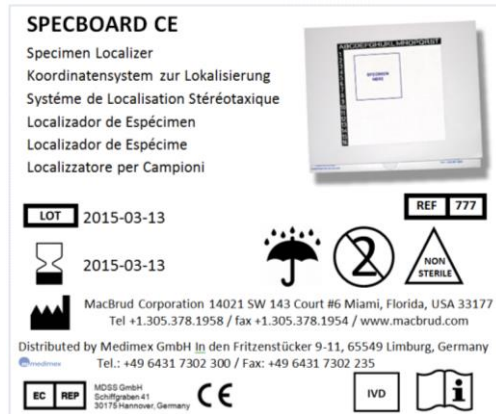

English: Biopsy Orientation Device

1. Mark on adjacent diagrams location of lesion and localizing wire.
2. Surgeon marks medial and superior margins prior to removal of specimen from breast.
3. Assistant holds "SPECBOARD" open & surgeon places specimen on upper left corner of grid with superior margin at the top.
4. Assistant closes "SPECBOARD" with resealable tape.
5. "SPECBOARD" placed in polyethylene bag, then sealed and X-RAYED.
6. X-RAY & "SPECBOARD" sent to Pathologist with Radiologist's interpretation.
7. Pathologist uses this sheet as right angle against X-RAY grid to determine location of lesion inside specimen.
8. Apparatus discarded in appropriate manner.

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This document is confidential and the property of MacBrud Corporation. Possession of this document indicates the holder to be under the confidentiality agreements of MacBrud Corporation
MacBrud Corporation, 14021 SW 143 court #6, Miami, Florida 33177, USA +1.305.378.1958,
+1.305.378.1954 Page 1 of 2

MacBrud Corporation – Instructions for use SpecBoard CE

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